EN Instructions for use

The Accu-Chek Softclix lancing device is an easy-to-use lancing device that has been specially developed to allow virtually pain-free collection of capillary blood from a fingertip or earlobe. There are 11 penetration depth settings so users can choose the one most suitable for their own skin type.

Only use the Accu-Chek Softclix lancets with the Accu-Chek Softclix lancing device since the use of other lancets may prevent or impair its proper function.

The Accu-Chek Softclix lancing device is intended for self-monitoring by a single person. It must not be used on more than one person owing to the risk of infection.

If the lancing device has been dropped while a lancet was inserted, the lancet may have worked loose in its holder. In that case it is no longer properly located and you cannot collect blood with it. In some cases the lancet may project from the opening in the cap and cause injury. Therefore avoid touching the front of the cap.

If the lancing device has been dropped, pick it up carefully and remove the cap from the lancing device. Beware of the lancet so as not to injure yourself. Always eject and dispose of the lancet.

Store the lancing device without a lancet inserted.

Do not store the lancing device at extreme temperatures (e.g. in a hot car).

Cleaning the lancing device

Wipe the lancing device once a week with a cloth moistened with 70 % alcohol or 70 % isopropanol (available in pharmacies). Also, wipe the inside of the cap with a cotton bud likewise moistened with 70 % ethanol or 70 % isopropanol. Allow the lancing device to dry thoroughly.

Do not immerse the lancing device in the cleaning liquid.

Guarantee

The statutory provisions on rights in consumer goods sales in the country of purchase shall apply.

Patents

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
EP-B-0565970; EP 1 034 740; US Re. 35,803; US 4,924,879; US 6,419,661
German Design Patent: DE 49900488.4

Notice regarding patents

Roche has been granted patent protection in the USA for its Accu-Chek Softclix lancing device and its use in combination with the Accu-Chek Softclix and Accu-Chek Softclix XL lancets (US Patents 4,924,879 and Re. 35,803). Similar patent protection also exists in other countries, especially through European Patents 0565970, 0782838 and Japanese Patent 2702374. Your purchase entitles you to use your Accu-Chek Softclix lancing device in combination with the Accu-Chek Softclix and Accu-Chek Softclix XL lancets. As required by statute, Roche will refrain from pursuing its patent rights against private persons using the Accu-Chek Softclix lancing devices in combination with lancets from another manufacturer for non-commercial purposes. Acquisition of the Accu-Chek Softclix lancing devices is not associated, other than through the aforementioned statutory limitation on the effect of the patent, with any entitlement and in particular with any tacit permission for the acquirer to use any other lancets. Roche reserves the right to assert its patent rights against any manufacturer offering lancets for the Accu-Chek Softclix lancing devices.

What the symbols mean

Various symbols are printed on the lancing device and the pack it came in. Their meanings are as follows:

-  Consult the instructions for use
-  For single use only
-  Use by/Expiry date
-  Sterilized using irradiation
-  Manufacturer
-  Catalogue number
-  Lot number
-  This product fulfils the requirements of Directive 93/42/EEC on medical devices.

 0088

Always use a new lancet each time you take blood. This decreases the risk of infection and blood collection remains virtually pain-free.

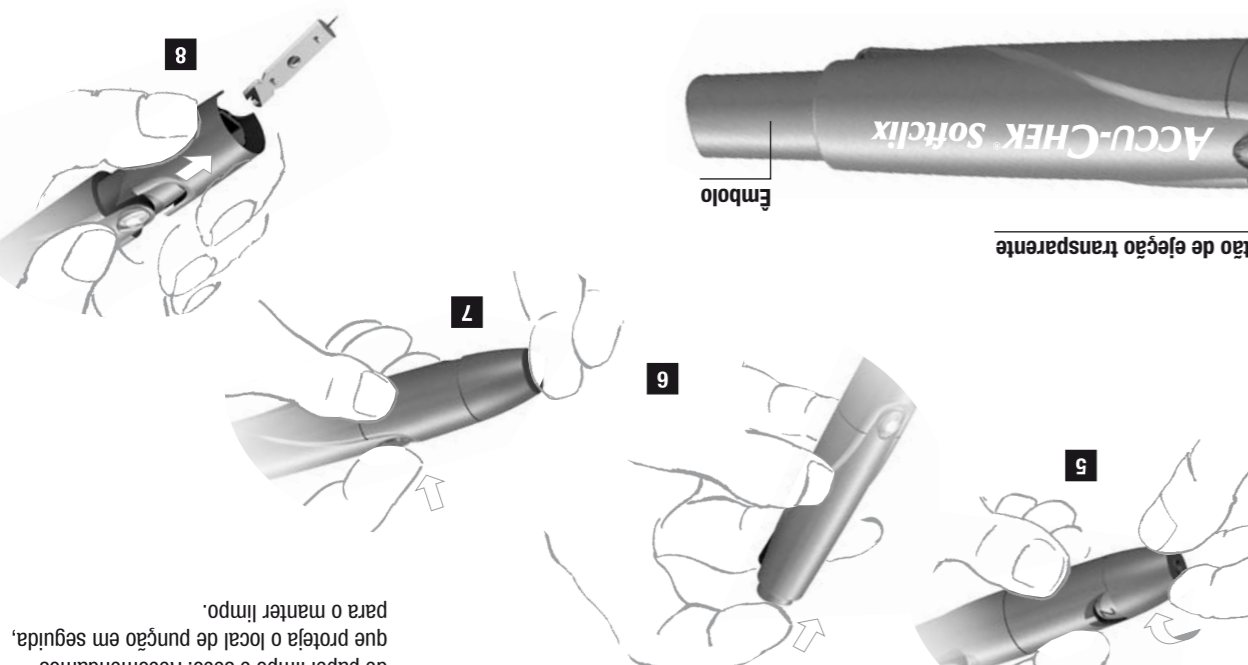
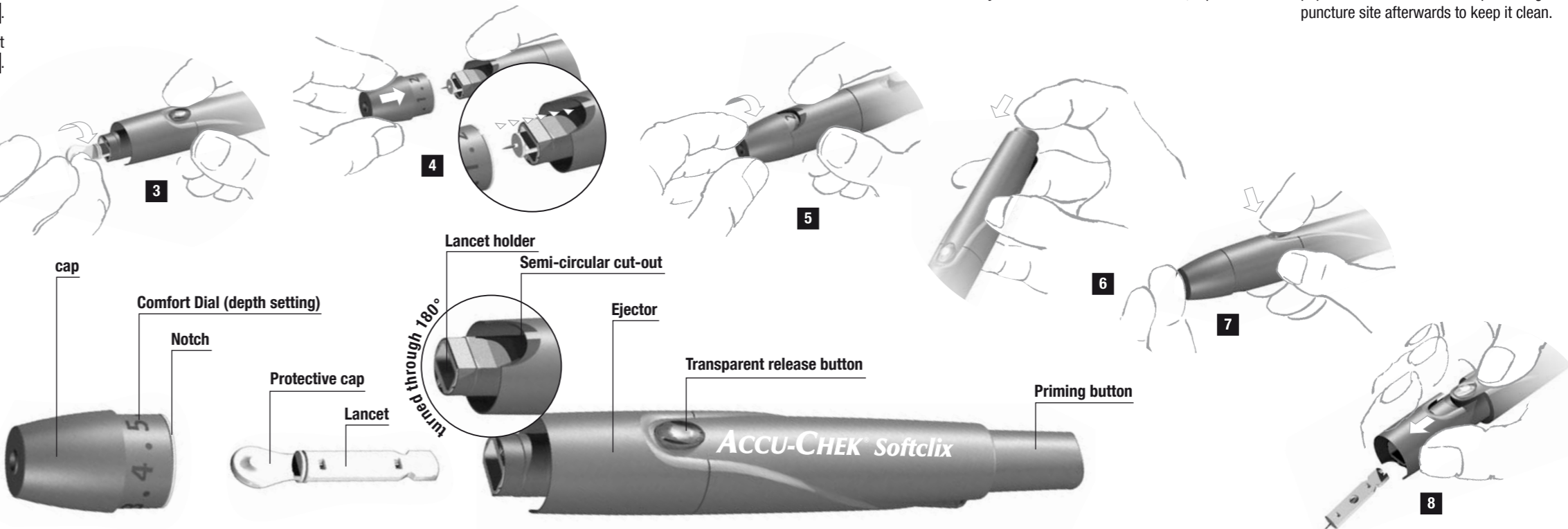
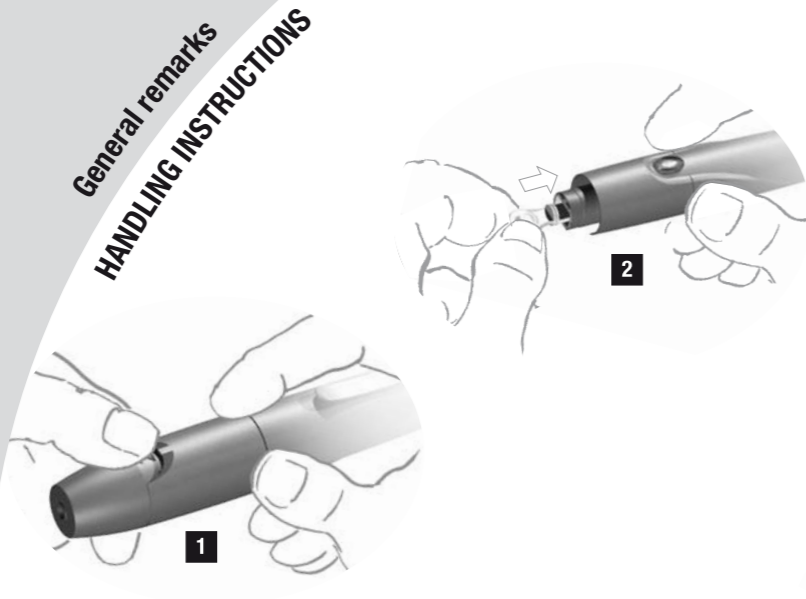
INSERTING THE LANCET

Pull off the cap from the lancing device **1**.

Insert a new lancet in the lancet holder and push it in until it clicks **2**.

Remove the protective cap from the lancet with a twisting motion **3**.

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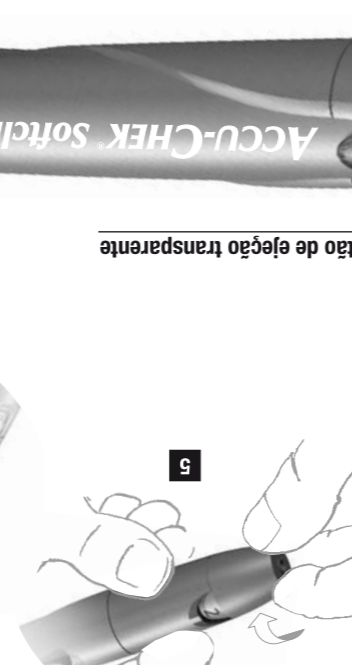
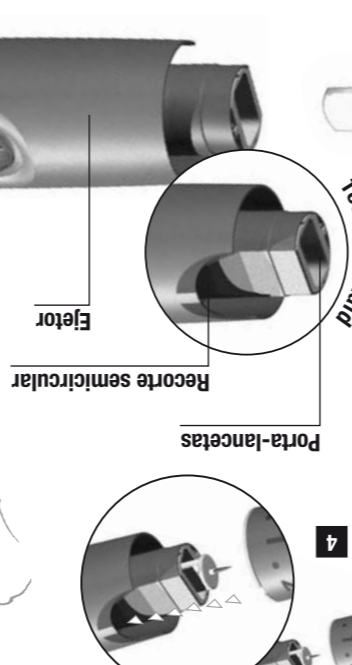
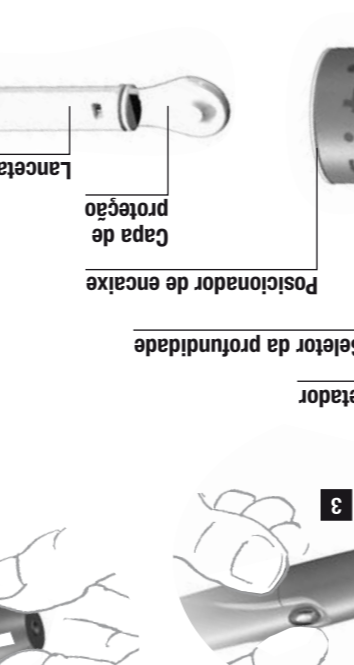
Ordering information

Accu-Chek Softclix kit (1 lancing device and 25 lancets): REF 0 3307450001
Accu-Chek Softclix lancets
25 lancets: REF 0 3307492001
200 lancets: REF 0 3307484001

Not all pack sizes will necessarily be available in all countries.

Last update: 2005-11

PREPARATIONS FOR TAKING BLOOD

All the fingertips and both the earlobes are suitable for obtaining blood. We recommend using the side of a fingertip, as this registers the least pain.

Wash hands with warm water and soap, and dry well. This helps ensure proper hygiene and stimulates blood flow.

SELECTING DEPTH OF PENETRATION WITH THE COMFORT DIAL

Modern blood glucose meters can take readings using just a tiny drop of blood.

Distributed by

Roche Diagnostics Ltd Charles Avenue, Burgess Hill, West Sussex, RH15 9RY, **United Kingdom**
UK Freephone number: 0800 701 000
R01 Freephone number: 1 800 709 600

Usually, enough blood can be obtained with a low depth setting. This reduces the pain of pricking the skin, after-pain is reduced or absent altogether, and tissue scarring is minimised.

The penetration depth is set by rotating the Comfort Dial (cap) **5**. If you are using the Accu-Chek Softclix lancing device for the first time, set the Comfort Dial to 1.

OBTAINING BLOOD

First check that you have set the desired penetration depth.

Press in the priming button until it clicks **6**. Be careful not to press the release button at the same time. The lancing device

Roche Diagnostics, 201 Boul. Armand-Frappier, Laval, Québec, H7V 4A2, **Canada**
Accu-Chek Customer Care 1-800-363-7949
Roche Diagnostics Australia Pty Ltd., 31 Victoria Avenue, Castle Hill, NSW 2154, **Australia**

ACCU-CHEK and SOFTCLIX are trademarks of Roche.
Roche Diagnostics GmbH
68298 Mannheim, Germany
www.accu-chek.com

procedure with progressively deeper settings. If too much blood emerges, reduce the penetration depth.

You are now ready to measure blood glucose according to the instructions that came with your meter.

EJECTING THE USED LANCET

Remove the cap from the lancing device **1**.

Slide the plunger forwards. The used lancet is ejected **8**.

Attach the cap again.

Dispose of used lancets so that the needles cannot cause injury.

SUBSEQUENT TREATMENT OF THE PUNCTURE SITE

Wipe the puncture site with a dry, clean paper towel. We recommend protecting the puncture site afterwards to keep it clean.

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